

# Creating a Test in Speaking Dynamically


## Drawing the Question and Answer buttons.

1. Click on the **cell tool** to draw your first sentence strip
2. Click on the **multiple cell tool** to duplicate your first sentence strip.
3. Click on the **pointer tool** to move or arrange your sentence strips
4. Click on the **cell tool** to draw your first answer button.
5. Click on the **multiple cell tool** to duplicate your first answer.
6. Select your answer cells with your **pointer tool**.
7. Click on the **Edit** menu and drag down to **Copy**.
8. Click on the **Edit** menu and drag down to **Paste** to duplicate your answer buttons.

## Question Button Functions

Each question button will have 2 features

1. Speak
  - a. Click on **Get Text**
  - b. Click on **OK**
2. Type Message
  - a. Click on **Get Text**
  - b. Click on **Insert Return**

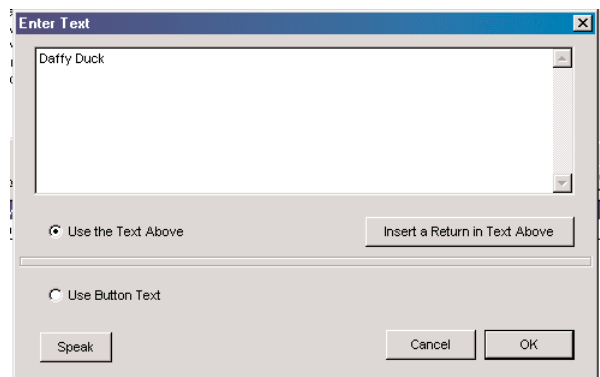


Assigned Actions	Action Details	Play
Speak Message	What is the state bird of Texas?	
Type Message	What is the state bird of Texas?	

## Answer Button Features

Each answer button will have 2 features

1. Spoken Preview
  - a. Click on **Get Text**
  - b. Click on **OK**
2. Type Message
  - a. Click on **Get Text**
  - b. Click on **Insert Return**

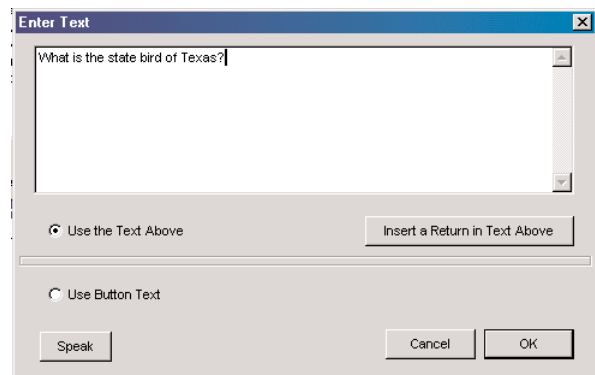

Assigned Actions	Action Details	Play
Spoken Preview	Daffy Duck	
Type Message	Daffy Duck	